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AESCULAP® „Surface“

For mid luminous conditions like
diagnostics, scaling, etc.

AESCULAP® „Rhodium“

For higher luminous conditions
like surgery.

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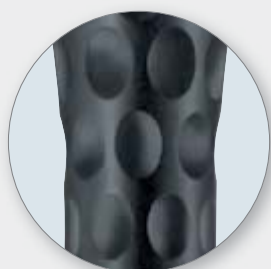
Make yourself comfortable.

You'll never want to put them down again – the new instruments in AESCULAP®'s Ergoprobe range make working that much easier, more relaxed and less tiring. The secret lies in the extremely lightweight, ergonomic handles with surfaces specially designed to have a particularly pleasant feel.

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Ergoprobe combines design with comfortable ergonomics and top functionality

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Good Functionality

- Dental probe working ends made of Phynox steel
→ highly gracile | elastic | dimensionally stable
- Periodontal probes are coated with TiAl-nitride
→ accurate marking | smooth transitions

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Comfortable ergonomics

- Shape and thickness ensure low-fatigue work
- Extremely lightweight handles

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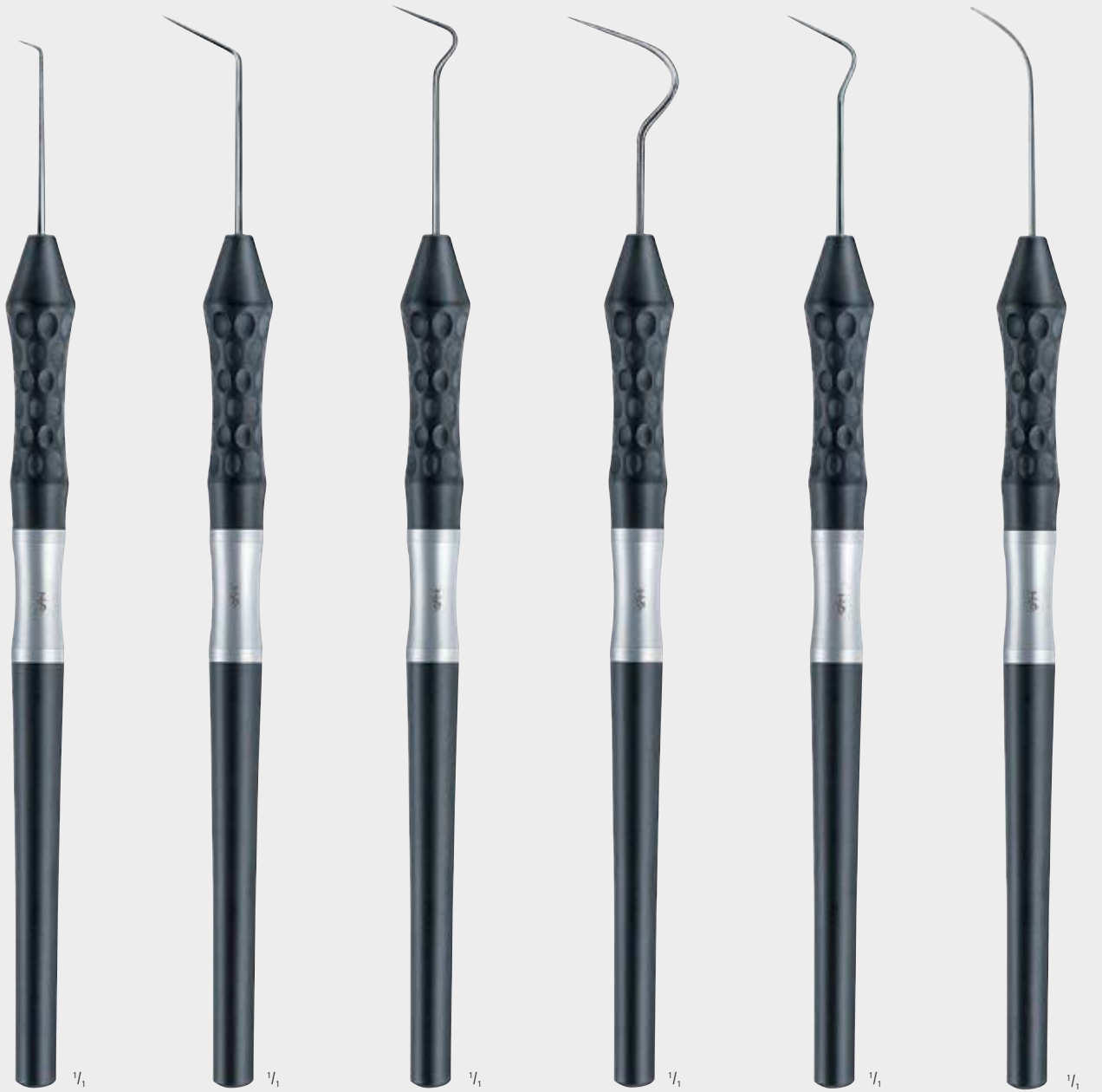
Aesthetics

- The winner of the iF DESIGN AWARD 2015
Category Medicine/Healthcare

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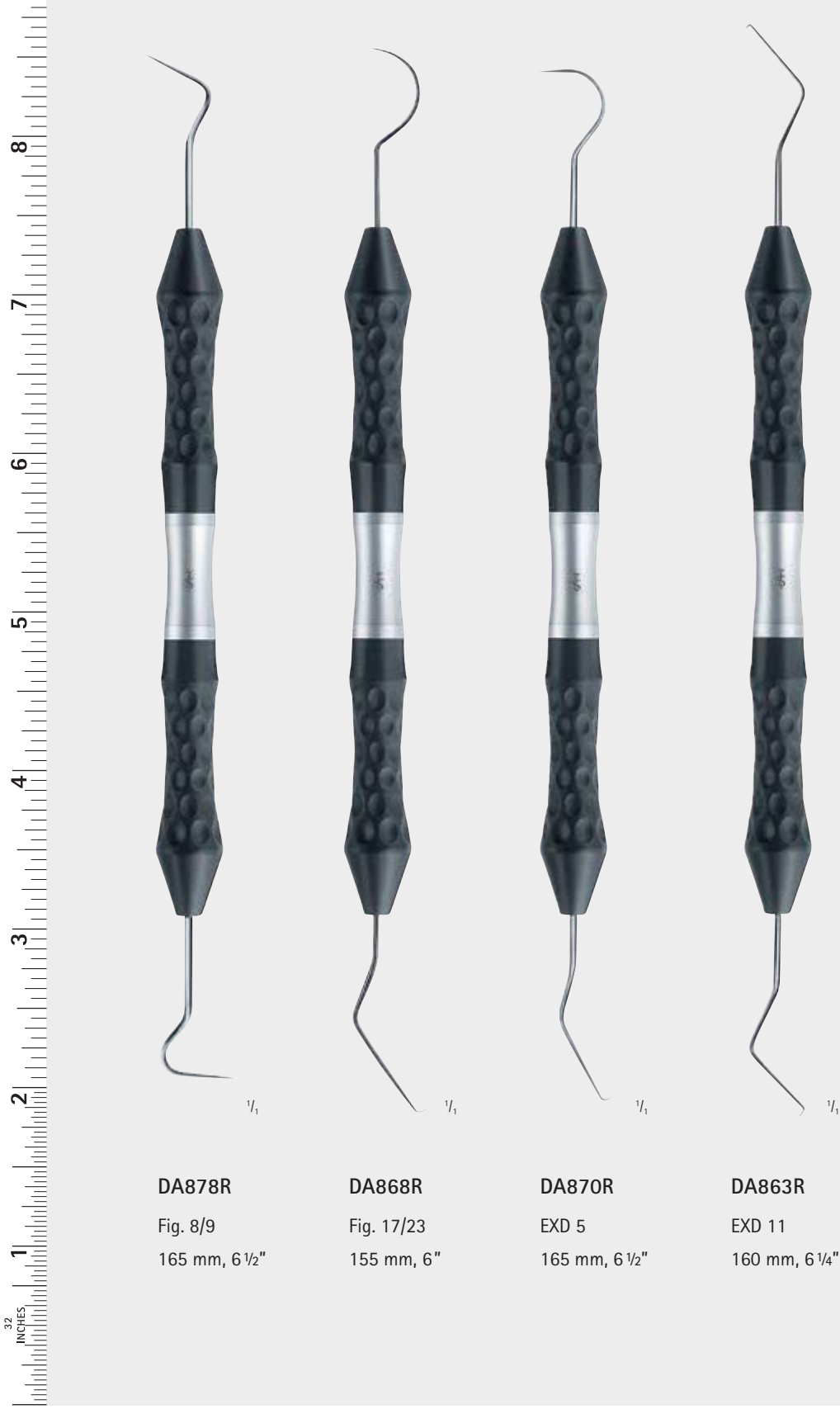
Due to a lack of measurement pressure standardisation, clinical measurements taken using different probing techniques often provide contradictory results. Objective evaluation of periodontal status is only possible when using pressure-sensitive periodontal probes. When assessing periodontal disease, probe measurements are subject to numerous instrumental and subjective influences, which can often result in underestimation of actual probing depths. When too little pressure is used, the main limiting factor on measurement probes is soft-tissue morphology rather than proportions of coronal, supra-alveolar fibre structures. Thanks to their defined spring pressure of 0.2 N, the DB764R and DB765R periodontal probes allow standardised probing in follow-up care of periodontal diseases, and especially in cases of inflammatory gingival changes in long-term studies.

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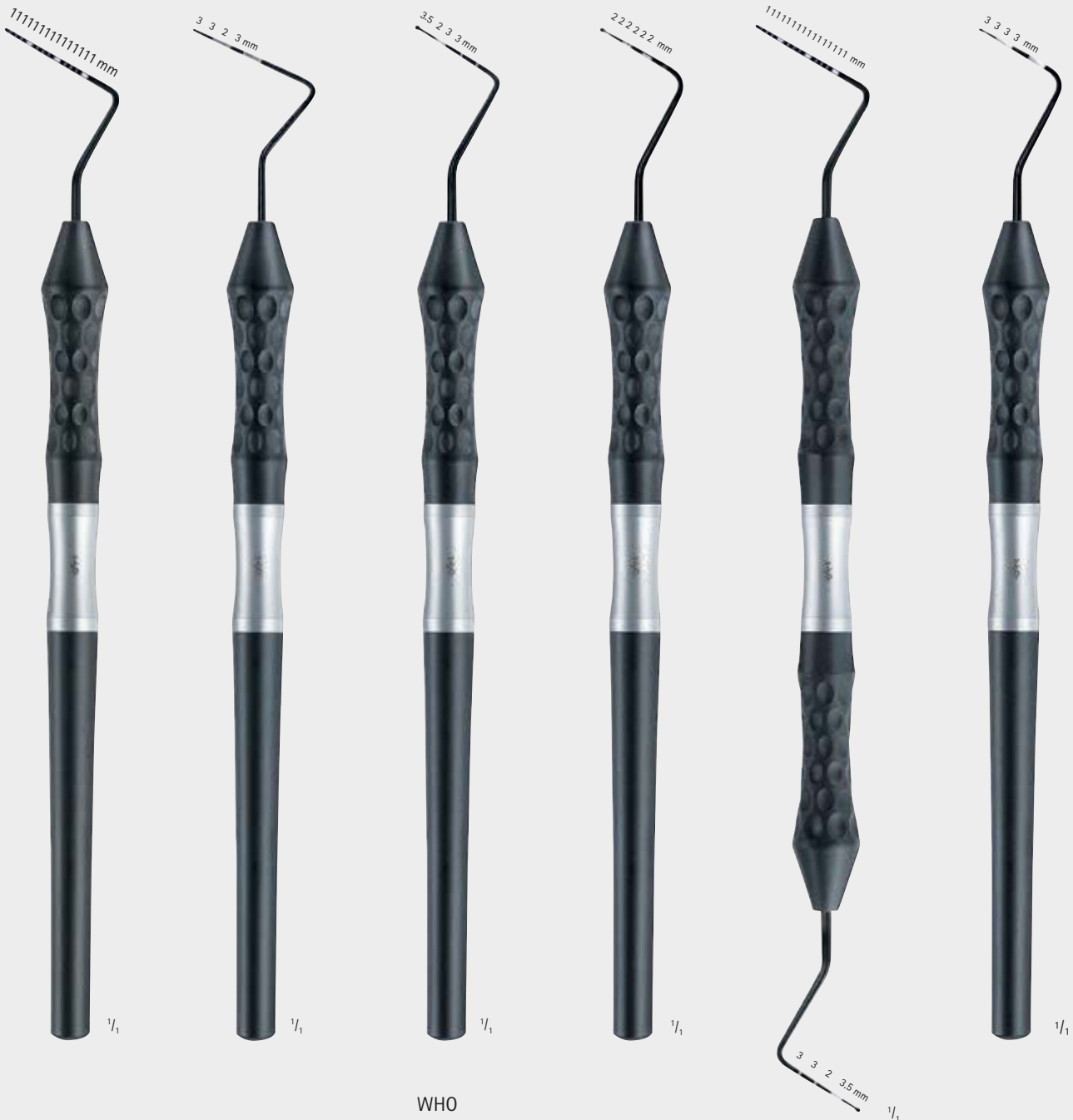
Colour coding system, coated with TiAl-nitride

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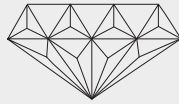
Special tooth tweezers LONDON-COLLEGE, mod.
Particularly strong, distortion-free design, serrated.

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DUROGRIP®-Tooth forceps with tungsten carbide insert for secure grasping of matrices and interdental wedges.

DA246R
Zahnpinzette
Dental forceps
150 mm, 6"

DA254R
150 mm, 6"